---

- **Urgent Field Safety Notice SmartConnector and Diamant v3.8 or v3.9**

FSCA-identifier: [bug 16035](#)

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---

Type of action: Notification of Resuscitation Policy not timely updated on the SmartConnector User Interface

Date: September 23, 2016

Our Ref: [VIG-6](#)

Affected device: SmartConnector, used in combination with Diamant version 3.8 or 3.9, all revisions. **If you do not use a SmartConnector at a dialysis station, this message does not apply to you**

Description of the problem:

From Diamant v3.8, the resuscitation policy as displayed on the SmartConnector can be out-of-date in case it was changed whilst the patient had already been selected by entering his key on a SmartConnector. Re-entering the key will not resolve the problem.

This may result in patients receiving - or NOT receiving - CPR - according to their policy. If unnoticed this may lead to serious patient harm/death. Or the patient may be successfully resuscitated against his/her will.

No actual occurrence of either possibilities has been reported so far.

Strong Advise on action to be taken by the user:

1. Inform staff of this undesired behaviour of the system.
2. Check resuscitation policy always on a Diamant Client (PC), by opening the latest Medical Patient Record where resuscitation policy is maintained.

Transmission of this Field Safety Notice:

This notice needs to be passed on all those who need to be aware within your organisation. Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action. This can be done by checking bug 16035 on our website.

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End of dutch translation

Transmission of this Field Safety Notice:

This notice needs to be passed on all those who need to be aware within your organisation. Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action. This can be done by checking bug 16035 on our website.

---

Contact reference person:  
Diasoft B.V.  
Jan van den Berge

The sender confirms that this notice has been notified the appropriate Regulatory Agency